



# Ingrowing Toenail Trainer

Part No: **80060**

This unique model closely mimics an inflamed toe end and allows for procedures such as wedge excision, ring block and total ablation of the nailbed.

## Skills

- Ring block techniques
- Application of a tourniquet
- Wedge excision
- Phenolisation techniques
- Ablation of the nailbed

## Features

- Cost effective: 3 Toe Ends in the set
- 2 wedge excision procedures can be carried out on each toe end
- Toe ends can be replaced in seconds

## Package supplied

- 1 Toe Jig **80071**
- 3 Toe Ends **80070**


## Attachment

**1**



Thread the flexible plastic strip attached to the toe through the slot at the front of the spindle on the toe jig.

**2**

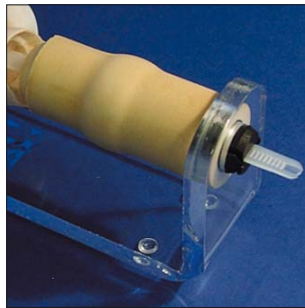


Push the plastic strip all the way through the spindle until it appears from the opposite end.

**3**



Grip the end of the strip and pull slowly until the toe end fits snugly against the front of the spindle. The toe assembly is now ready to use.

**1**

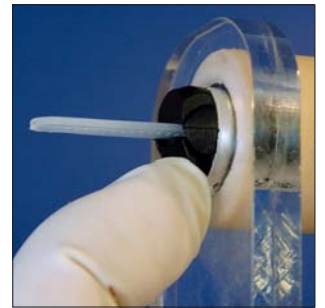

Depress the clip at the back of the spindle, underneath the plastic strip, and gently pull the toe end until it is clear of the toe jig.

## Care of products made from Latex Foam Rubber:

Latex foam is a natural product; store away from strong light and in an airtight bag to reduce exposure to ozone. Ozone is generated by some electrical equipment including telephones and computers.

Wash your hands before touching the foam. (If the foam is handled after contact with certain metals such as copper, it becomes yellow and discoloured).

The foam may be washed effectively using any soap and rinsed afterwards. 'Vanish'™ soap is very effective.

Do not ingest the product and practice normal hygiene after handling the products.



Designed and manufactured by Limbs & Things Limited,  
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.  
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501  
sales@limbsandthings.com www.limbsandthings.com

Issue 2, April 2003  
© Limbs & Things 2003